



#### English

1. To determine the mounting positions on the floor, place the support (052.7255) at the desired position in front of the wall. Make sure the distance between the wall and the support is **not less than 48mm nor larger than 75mm!** Then fix the support to the floor.
2. To determine the mounting positions for the wall bracket, mount the lifting column (A) together with wall bracket (B) **temporarily** on the support. For instructions see the manual of the wall bracket (052.7227).
3. Mount the lift column permanently to the support and fix the wall mount to the wall.

#### Nederlands

1. Plaats de steunvoet (052.7255) op de gewenste positie voor de wand en bepaal de montageposities op de vloer. Zorg er voor dat de afstand tussen de wand en de steunvoet **niet kleiner dan 48mm noch groter dan 75mm is!** Bevestig vervolgens de steunvoet aan de vloer.
2. Monteer de hefkolom (A) met muurbeugel (B) **tijdelijk** op de voetsteun om de montagepositie van de muurbeugel te bepalen. Zie hiervoor ook de handleiding van de muurbeugel (052.7227).
3. Monteer de hefkolom (A) met muurbeugel (B) definitief op de voetsteun en bevestig de muurbeugel aan de muur.